






# belletto

## A. Manual control unit

1. Back section up 
2. Back section down
3. Upper leg section up 
4. Upper leg section down
5. Comfort position 
6. Height adjustment up 
7. Height adjustment down
8. Caster button\* 

\* Pushing the caster button (8) at the same time as the “height adjustment down” –button (7) extends the casters and allows the bed to be moved.

# belletto


## B. Side rails

To raise a side rail section, pull the section horizontally out of its location to end-stop and fold it upwards. In order to adjust the height of the side rail, pull the telescopic part up to the stop.

To lower a side rail section completely,



Press the two buttons on the inner side of the frame, right above the cross-member, so that the height-adjustable side rail section can be put into its lowest position. (fig. 1)

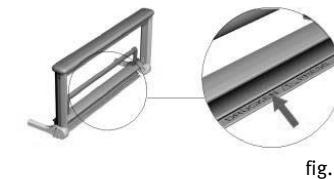

Operate release button labelled “Push” at the bottom of the side rail part. (fig. 2)



tilt it horizontally to the side so that it runs parallel to the floor. (fig. 3)



Push the side rail completely under the lying surface. (fig. 4)